



ONTARIO MINE RESCUE
PREPARED SINCE 1929

CAREVENT POST USE SERVICE

Team	Date:
Team Member:	

Demerit: 2 demerits for each omission, 1 demerit for wrong order

For Failure to:	Demerit
1 Remove and wash mask, connector and diaphragm	
2 Wipe resuscitator and hose with germicide or clean cloth	
3 Install two full cylinders, ensure to check the hydrostatic date	
4 Ensure the o-ring is lubricated	
5 Inspect exhalation valve	
6 Re-assemble resuscitator	
7 Ensure the automatic cycle functions	
8 Press the CPR button and ensure the automatic cycle function resumes	
9 Test the Ventilator Limiting Pressure function	
10 Ensure to de-energize the system	
11 Inspect the face mask, apply anti-fog and place in a bag	
12 Repack the CAREvent	
13 Seal the case, dated 6 months in the future	

Total Demerits:

--

Signature of Judge

--